Aston

DUSCH- TOILETTENROLLSTUHL

WHEELED COMMODE

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Foreword

We congratulate on the buying of a technically matured and the best available technology product.

This instructions helps you and the carers as a basis for sure, safe use of the toilet wheel chair and a sensitive use of the product's possibilities.

The instructions describes the preparation, service, use, servicing and cleaning of the product.

This instructions is to be perused carefully and must be always accessible to all users of the product.

Indication

Shower-Toilet wheelchairs are necessary when using or achievement of the toilet is not possible because of a handicap. Shower-Toilet wheelchairs are also usable as room wheelchairs. Shower-Toilet wheelchairs can be pushed over customary toilet bowls.

Contraindication

The wheeled commode is not suitable for persons

- with reduced or impaired ability to sit
- with significantly impaired cognitive ability

Safety instructions

- Read this instructions carefully.
- Before use all components are to be checked for damages and firm seat.
- While seating, rising and while inserting the toilet bucket as well as folding up or engaging the armrests it's possible to pinch clothing or body Parts.
- Always apply the parking brake when using the shower-toilet wheelchair.
- Before getting in/out from the shower-toilet wheelchair swing away the footrests.
- Never use the footrests as stand up device - Tilting danger!
- Protect from unauthorized use, e.g., by children, by locking the brakes

Reapplication

This product is suitable for reapplication.

With the processing the standards of the manufacturer must be considered. This standards are made available on inquiry.

Assembly

Insert the four 5" wheels in the admission pipe in the frame in the desired position and fix them with the provided bolt clips. (See picture 1+2).

Put the premounted backrest on the basic rack and fix it with the star screws with a clockwise rotation. (See picture 3).

Locate the seat by sliding the back of the seat in to the frame. Then press down on the whole of the seat until secure. Add the aperture cover if required.

Insert the bucket from the rear side into the guiding under the seat plate, after use pull out also from the rear side. (Picture 4).

Attach the footrest by placing the top bracket in to the receiver tube at the top of the frame, ensuring the footrest is swung back. (Picture 5).

Swing the footrest towards the front of the chair until it locks in position. (Picture 6).

Repeat for the other footrest and adjust the footplate height and angle if required. (see instructions).

Assembly and disassembly of 24" - wheels

The wheels have quick release pins as axles. Press the button on the quick release pin and insert through the centre of the wheel and then insert the wheel and axle into the bracket mounted to the chair. After installing, release the button and check the wheels are secured on to the frame. Repeat for the other wheel.

Install the brakes in the position shown overleaf, ensuring that the brakes operate securely, correctly and easily.

Castor Brakes

To lock the castor press down on the locking tab with a foot (as shown). To release press the other side of the locking tab with a foot. Unless the chair is fitted with 24" wheels, it is recommended that the castors are always locked when the chair is stationary.

Swinging Footrests In and Out

The footrests can swing in and out if the user requires this. If required, first flip up the footplates.

Then, press the lever to unlock the footrest whilst rotating the footrest in the required direction.

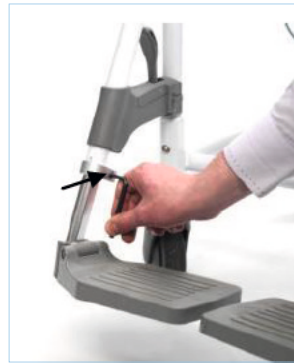
When required, rotate the footrests to the forward position until they automatically lock in position. Flip down the footplates if required.

Footplate Angle Adjustment

To adjust the footplate angle, slacken the bolt near the bottom of the inner tube (chrome). Adjust the footplate to the required angle and resecure the bolt.

Footplate Height Adjustment

To adjust the height of the footplate, slacken the bolt at the end of the white tubing of the leg rest, using an allen key and turning the bolt anticlockwise. Slide the footrest to the required height and secure the footrest by retightening the bolt. Tighten the bolt by turning clockwise with an allen key.

Cleaning/Maintenance

- Clean regularly by hand with a damp cloth and usual household cleaning agents!
- Use spraying disinfectants p.e. Descosept AF or Bacillol AF.
- Do not use any aggressive detergents!
- Screws must be checked regularly and if necessary retightened.

Type label

- 1 Product name
- 2 Article number
- 3 Serial number
- 4 Date of manufacturing year/month
- 5 Attention: See operating instructions

Warranty

We issue a warranty of 24 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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