# BEN

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## Table of contents

Preface.....	7
Declaration of conformity.....	7
Service life.....	7
Disposal.....	7
Identification plate.....	7
Reuse.....	7
Warranty.....	7
Parts of the product.....	8
Indication.....	8
Scope of delivery.....	8
Safety instructions.....	8
Securing in the bathtub.....	9
Swiveling the seat.....	9
Use.....	9
Cleaning   Maintenance.....	9
Technical data.....	9

## Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

## Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co.KG declares compliance according to the medical device regulation (EU) 2017/745.

## Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

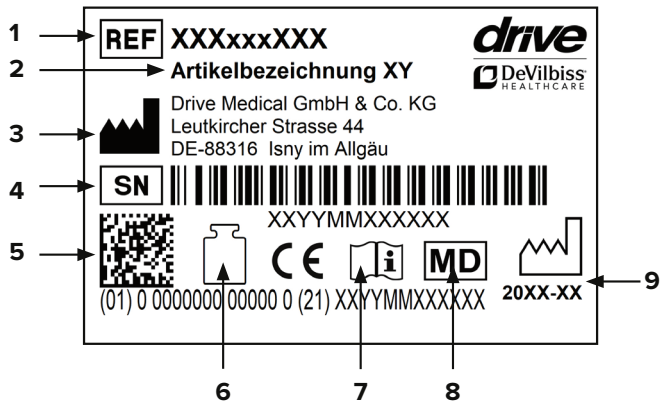
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

## Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

## Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year/month)

## Reuse

This product can be reused. Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available at [www.drivedevilbiss.de](http://www.drivedevilbiss.de) or from your specialist dealer.

## Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

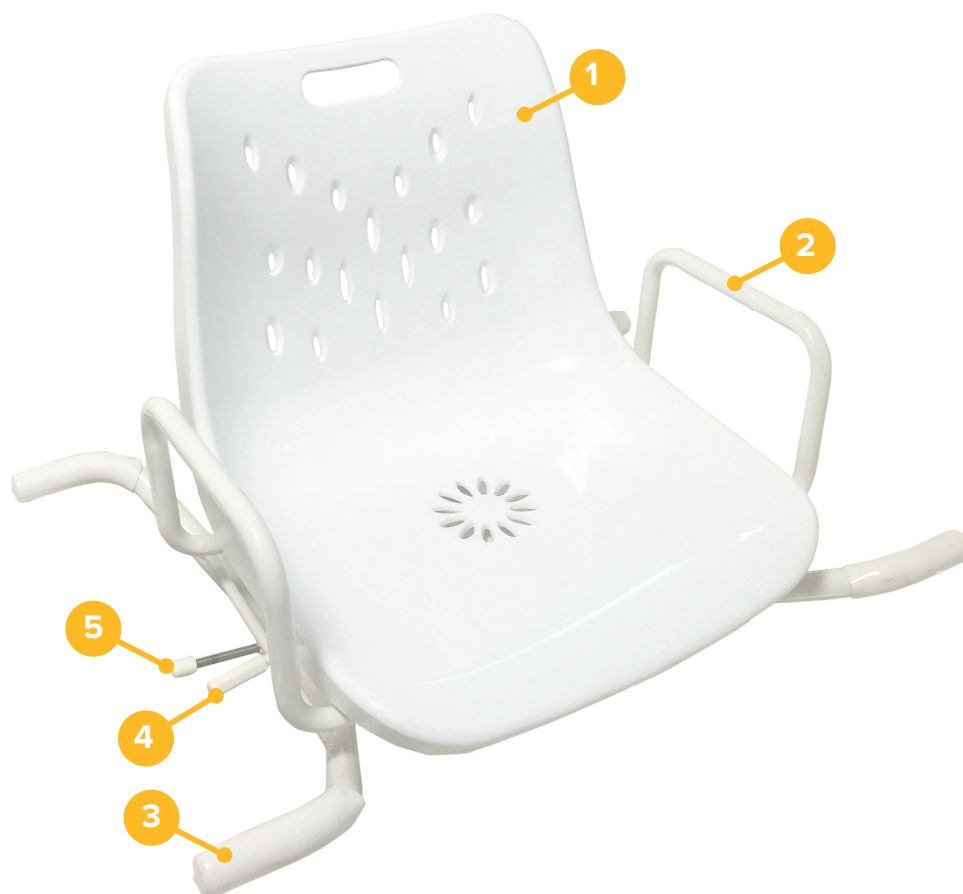
Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

We reserve the right of error and modifications on the basis of technical improvements and changes in design.

### Exclusion of warranty

Parts subject to wear include (among others):  
Protective cap/tub support

## Parts of the product

BEN

- |                                      |                         |
|--------------------------------------|-------------------------|
| 1. <b>Seating and swiveling unit</b> | 4. <b>Release lever</b> |
| 2. <b>Armrests</b>                   | 5. <b>Fixing screw</b>  |
| 3. <b>Protective cap/tub support</b> |                         |

### Indication

#### INDICATION:

Impossibility to be securely seated or sit down and/or impossibility to maintain seated position and, if applicable, limitation in independently getting into/out of the bathtub and/or due to significantly reduced function or lack of function of the lower extremity with limited or absent mobility and muscle strength and/or significantly limited control of the torso with simultaneously preserved adequate residual functions of the upper extremities. For use in home care.

### Scope of delivery

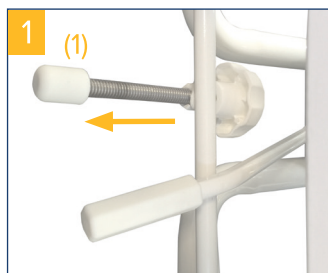
1 x	Swiveling bath seat
1 x	Instructions for use

### Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Proceed slowly and carefully when sitting down. Do not drop!
- Caution – risk of slipping/falling!
- Do not stand on the bath seat.
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.


## Securing in the bathtub

Rotate both set screws **(1)** until they touch the edge of the tub and prevent the seat from slipping (**pic. 1**).

## Swiveling the seat

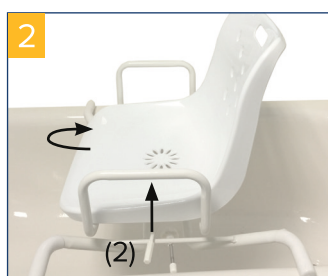The seat can be swiveled after pulling the locking lever **(2)** (**pic. 2**).

This function should be checked prior to initial use.

## Use

- Hold onto the armrests securely with both hands and sit on the seat.
- Now pull the locking lever **(2)** upwards, raise your legs and swivel them into the tub (**pic. 2**).
- The released locking lever automatically clicks into all 90° positions. Do not press the locking lever downwards!
- To get out of the tub, follow the steps in reverse order.

## Cleaning | Maintenance

- Clean regularly with a moist cloth!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60 °C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

## Technical data

Article number	550600100
HMV (medical aids register) no.	04.40.02.3012
Max. load	120 kg
Seat width x seat depth	46 x 42 cm
Overall width	72 cm
Height of backrest	42 cm
Seat adjustment	4 x 90°
Color	white
Total weight	6.1 kg
Frame material	Steel, powder-coated
Seat material	Plastic (PE-HD)

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Drive Medical GmbH & Co. KG